

PARI BOY® family

Efficient inhalation systems – as individual as the patient's needs

- with PARI LC SPRINT nebuliser – clinically tested in over 50 scientific studies
- efficient lung deposition even with short inhalation times
- with PIF Control – the integrated “training tool” helps patients to learn an effective inhalation technique



PARI BOY® Junior
for babies and small children



PARI BOY® Classic
for use in acute and chronic diseases of the airways of patients aged 4 years and older



PARI BOY® Pro
for use in severe and chronic diseases of the airways of patients aged 4 years and older

At a glance



PARI BOY Junior
Item No. 130G1301, Western Europe (de, fr, it, pt, es, en)
Item No. 130G1306, Eastern Europe (el, hu, pl, sl)



PARI BOY Classic
Item No. 130G1201, Western Europe (de, fr, it, es, nl, en)
incl. child and adult mask
Item No. 130G1206,
Eastern Europe (bg, el, hr, pl, ro, sl, et, lv, lt)



PARI BOY Pro
Item No. 130G1001,
Western Europe (de, fr, it, pt, es, nl, en)
Item No. 130G1006, Eastern Europe (cs, el, hu, pl, sk, sl)

PARI Year Packs



PARI BOY Junior Year Pack
Item No. 023G3311,
Western Europe (de, fr, it, pt, es, nl, en)
Item No. 023G3316,
Eastern Europe (bg, cs, el, hr, hu, pl, ro, sk, sl, et, lt, lv)



PARI BOY Classic Year Pack
Item No. 023G3211,
Western Europe (de, fr, it, pt, es, nl, en)
Item No. 023G3216,
Eastern Europe (bg, cs, el, hr, hu, pl, ro, sk, sl, et, lt, lv)



PARI BOY Pro Year Pack
Item No. 023G3011,
Western Europe (de, fr, it, pt, es, nl, en)
Item No. 023G3016,
Eastern Europe (bg, cs, el, hr, hu, pl, ro, sk, sl, et, lt, lv)



PARI GmbH
Moosstraße 3
82319 Starnberg, GERMANY
Tel.: +49 (0)8151 279 220
Fax: +49 (0)8151 279 6220
E-Mail: export@pari.de
www.pari.com

130D0030-A-2019-09



Nebuliser therapy at home



PARI BOY® family – Tips for use and hygiene

www.pari.com

Usage tips


- 1 Wash your hands thoroughly before each session.



- 2a Attach the mouthpiece or the mask to the nebuliser.



- 2b The baby mask must only be used in conjunction with the baby bend. Adjust the alignment of the bend and the mask according to the position the baby or infant is in.

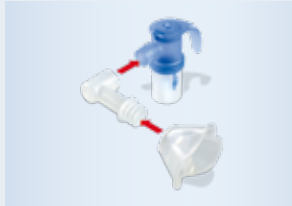

- 3 Connect the nebuliser and compressor with the connection tubing. Connect the compressor to the mains power supply.



- 4 Open the nebuliser cap. Pour in the inhalation solution. Close the nebuliser cap.



- 5a Sit upright in a relaxed position and switch on the device. Take the mouthpiece between your teeth and close your lips around it.



OR

- 5b Make sure that the nebuliser is upright. Gently press the mask against the face so that it fits snugly.



- 6 Breathe in slowly and deeply. When breathing in too fast, PIF Control produces a resistance. If you notice this, stop for a moment, then try to inhale more slowly. When a different noise comes from the nebuliser, this indicates that nebulisation has finished.



- 7 Switch off device. Disconnect the nebuliser from the tubing. Switch the device back on to dry the tubing.



- 8 Disconnect the power plug.



Hygiene tips

1. Disassemble the nebuliser after each use and **wash** all individual parts in warm water with dishwashing liquid, before rinsing thoroughly in running water.



2. **Disinfect** the nebuliser once a day for 5 minutes in boiling water or for at least 6 minutes in a standard commercial disinfectant, such as one designed for baby bottles.




3. After every cleaning and/or disinfection, place the parts on a dry, clean, absorbent surface and allow to **dry** completely.



4. **Store** all components them in a dry, dust-free place.



5. Air filter, nebuliser and connection tubing should be **replaced** every year. These components are included in the **PARI Year Packs**, which are available from your PARI dealer. Components may have to be replaced sooner if they are damaged or if contamination cannot be removed.

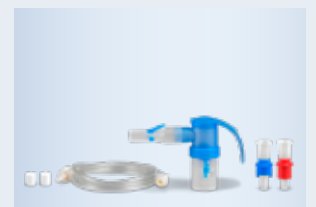

Quick Reference Guide
For more details, please
read and follow
the Instructions for Use.