

## Prepzyme®

Item No. 34559

### **Multi-Tiered Enzymatic Foam Spray**

### **Helps to Prevent The Adhesion Of Bio-Burden To The Surfaces of All Surgical Instruments and Scopes**

Prepzyme® is a safe and ideal method for keeping soiled instruments and scopes moist, helping to prevent bio-burden from adhering to the surface prior to decontamination. Prepzyme® covers the instruments and begins breaking down blood, fat, protein and carbohydrates before they arrive in the central service/decontamination room.

Perfect for use in Operating Rooms, Endoscopy Suites, Outpatient Surgery, Dental and other departments before instruments are transported to decontamination. Prepzyme® prevents splashing during transportation, meaning no more messy spills. By eliminating dried on bio-burden, it helps reduce cleaning time. Prepzyme® is clinically tested to be free rinsing and 100% biodegradable.



### **Features**

- Instantly active - Constantly active
- Begins breaking down blood, fat, protein and carbohydrates
- No more messy spills or splashing
- Helps to prevent dried on bio-burden
- Easy and safe to use
- Safe for all instruments and scopes - Extends usable lifetime
- Neutral pH (non-abrasive)
- 100% Biodegradable
- Free rinsing
- Will not harm any metals, plastics, rubber or corrugated tubing

### **Directions for Use**

1. Spray directly on soiled instruments and scopes immediately after use.
2. Allow foam to stay on instruments and scopes until ready for cleaning.  
Apply more Prepzyme® as needed to keep moist.
3. Rinse instruments thoroughly before cleaning.

Discard used solution.

Cautions: Avoid prolonged contact with skin. Do not swallow. Keep away from children.